

Multi-Clin™

# Multi-Clin™

11-Panel Cassette

1 Sample • 5 Minutes

# 11 Results



- **11 Analyte Drugs of Abuse Point-of-Care Panel**

- *Amphetamine, Barbiturates, Benzodiazepines, Cocaine, Marijuana, MDMA, Opiates, Oxycodone, Phencyclidine, Propoxyphene & Tricyclic Antidepressants*

- **8 HOUR RESULT STABILITY**  
*Negative Results will not turn Positive*

- **One Step Test**

- **Results Within 5 Minutes**

- **Built-in Positive AND Negative Procedural Controls**

- **24-month Shelf Life at Room Temperature**



The Multi-Clin™ 11- Panel Drug Screen shown above.

Clinical Point-of-Care

# Drugs of Abuse Testing

Brands You Can Trust™







# Multi-Clin™ Clinical Panel Drugs of Abuse Test



- 8 HOUR RESULT STABILITY
- Built-in Positive and Negative Controls
- Complete Analyte Menu
- One Step Procedure
  - Add urine to sample wells
  - Read results within 5 minutes

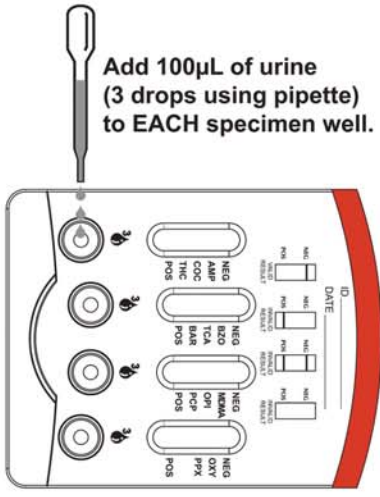


Wait 5 minutes to read results.

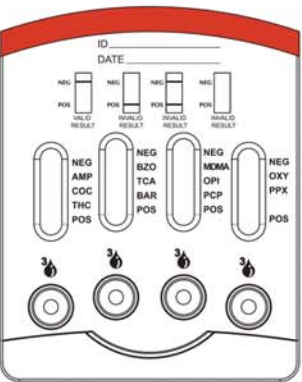

Confirm validity of results by checking NEG and POS Control Regions.

There should be NO line in the Positive Control Region (POS) and a visible line in the Negative Control Region (NEG).

Interpret Results at 5 minutes.

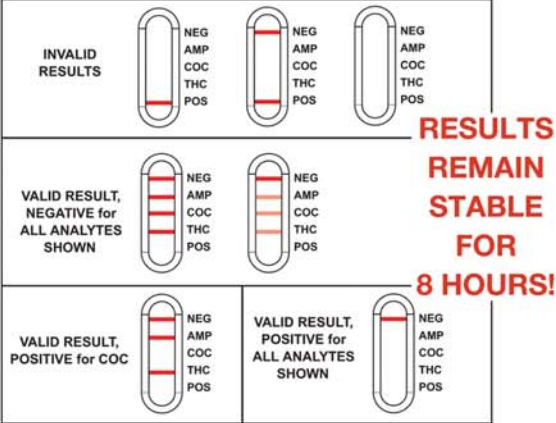

## Custom Configurations Available

Available Analytes & Cut-off Levels			
Amphetamine (AMP)	1,000 ng/mL	Opiates (MOP)	300 ng/mL
Barbiturates (BAR)	300 ng/mL	Oxycodone (OXY)	100 ng/mL
Benzodiazepines (BZO)	300 ng/mL	Phencyclidine (PCP)	25 ng/mL
Cocaine (COC)	300 ng/mL	Propoxyphene (PPX)	300 ng/mL
Marijuana (THC)	50 ng/mL	Tricyclic Antidepressants (TCA)	1,000 ng/mL
Methylenedioxymethamphetamine (MDMA)	500 ng/mL		