



Instructions

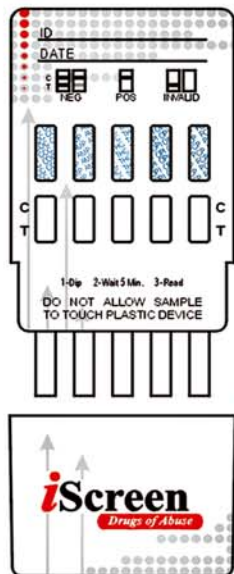

1. Remove iScreen™ test from foil pouch at room temperature. Take off the cap revealing the testing strips.
2. Immerse test vertically in the urine sample for at least 10 - 15 seconds. Do not allow the specimen to touch the device housing.
3. Read at five (5) minutes.
Results stable up to four (4) hours.

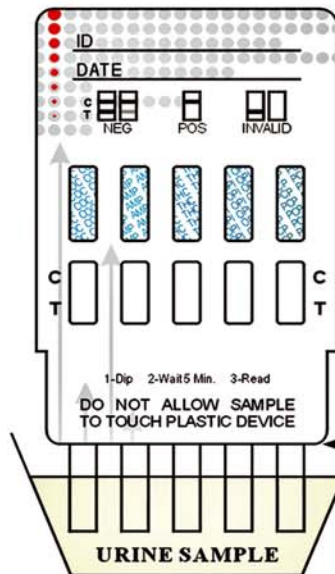

This illustration shows
a 5-Drug Test Card

Do not allow specimen to
touch the device housing

Interpretation of Results:



Negative

(2 Lines - Any indication of a line in both test and control fields is to be interpreted as a NEGATIVE)



Non-Negative

(1 Line)



Invalid

(No line(s) or no control line(s))